

## “AccuFit” MRI Headrest Drape Instructions For Model # BSM1003-01

The “AccuFit” MRI Headrest drape is designed to protect your equipment and patients from any oils, make-up or contact skin issues. This headrest cover will fit the following headrests:



Headrest #1



Headrest #2



Headrest #3



Headrest #4

1. Open with the elastic ends matching the form of the headrest if you have headrest #1. Start with the top and wrap around the ends. **FIG. 1**



Headrest #1 FIG.1

2. Pull the drape around the entire headrest as shown in **FIG. 2**



FIG. 2

3. For headrest #2, #3 and #4 make sure the elastic is down and the flat portion on top. Start the drape at the horseshoe end. (**FIG. 3**) Keep it level and pull it across the top around to the other end.



FIG.3

4. Adjust as necessary. Make sure you have complete visualization of the mirror for the patient.

