

“AccuFit” MRI Coil Drape Instructions For Model # BSM1004-01

The “AccuFit” MRI Coil drape is designed to protect your equipment and patients from any oils, dirt or contact skin issues. This coil drape is compatible with these coils:

Philips® Achieva 1.5T GE® Excite 1.5T
Philips® Intera 1.5T GE® Excite HD 1.5T
Philips® Achieva 3.0T GE® Excite HDx 1.5T
Philips® Intera 3.0T GE® Signa LX 1.5T
May fit coils other than those listed



Special features are designed into this drape. Occasionally due to lesion accessibility, the coil pad is removed. To accommodate this situation the Coil cover is lined with extra fabric for patient comfort.

1. Open the package. The top of the drape will align the round holes to match the coil holes.



2. Pull elastic around all edges for a secure fit.

NOTE:

The inner padding allows the coil pad to be removed and the AccuFit Coil drape to be utilized. The padding faces down next to the coil.

