

# *Instructions for the Ocelco Pillcrusher™*

To meet your needs for properly crushing various pills, the following three steps must be followed for maximum crushing efficiency.....


1) Place pills between two ¾ oz. paper souffle cups.

Note: DO NOT use plastic souffle cups, they could leave plastic slivers in the medication.

2) Pull handles together with only enough pressure to crush the pill.

Note: Too much pressure will re-compact the pill to wafer form.

3) After the initial crush, move the handle up and down lightly while revolving the cups. This combined motion will reduce the pills to a consistency similar to that of granulated sugar.

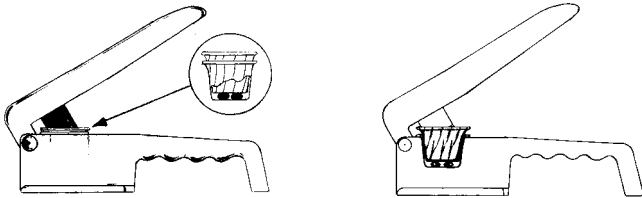

The handle pressure required will vary depending on the hardness of the pill. Soft tablets will require a softer handle pressure than a hard tablet, etc.