


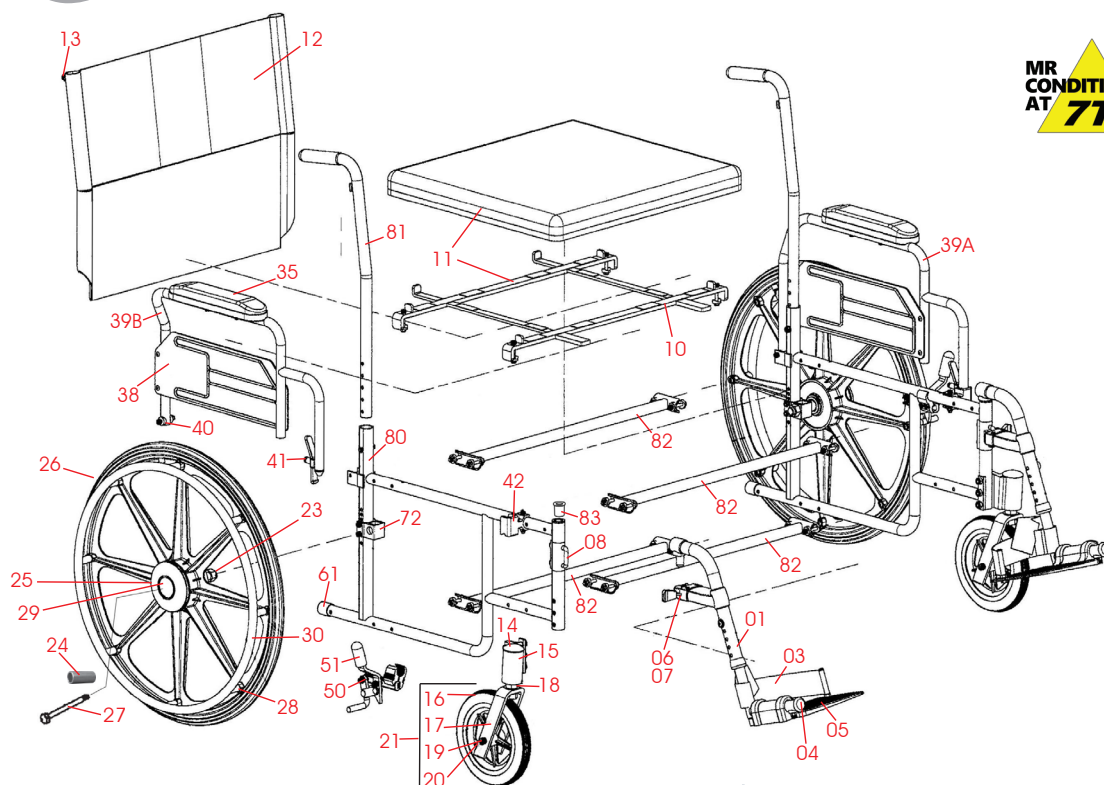

24" Wide HD Aluminum

WCA2425SD27

THIS ITEM PASSES

FERROMAGNETIC DETECTION SYSTEMS



24" Wide Aluminum Non-Ferromagnetic Wheelchair //

Fig	Model	Description	Page
01	FR3202L-05 FR3202R-05	Footrest Complete, Left Side * Footrest Complete, Right Side *	13
03	AC1602L-05 AC1602R-05	Heel Loop with Hardware, Left * Heel Loop with Hardware, Right *	23
04	LR7016X-01	Front Bumper, Black	16
05	FR3111X-01	Footplate Only for Footrest & Legrest Universal Left or Right	16
06	FR3210X-05	Front Rigging Lock Assembly Complete, Universal Left or Right	15
07	CA8015X-05	Spring Only, for Lock Assembly	15
08	FM1010X-05	Front Rigging Lock "U" Bolt	15
10	UP1430X-05	Seat Attaching Bracket Only	15
11	UP1403X-05-CL	Solid Seat Uph. w/Bracket **	18
12	UP1406X-05-CL	Back Upholstery w/Hardware **	18
13	UP1224X-01	Back Attaching Hardware Set	19
14	WH3133X-05	Black Journal Cap	9
15	WH3115L-05 WH3115R-05	Fork Caster Assembly, Left * Fork Caster Assembly, Right *	6
16	WH3116X-05	8" Wheel with Bearings	5
17	WH3118X-05	Fork with Nut	6
18	BE1051X-05	Bearing for Fork Stem	4
19	WH3119X-05	Axle and Nut for Front Wheel	8
20	BE1017X-01	Bearing for Front Wheel	4
21	WH3104X-05	Wheel Assembly, Fork, Wheel, Bearings and Axle	6
23	WH3137X-05	Spacer, Between Wheel & Frame	7
24	HW9215X-01	Spacer, Between Wheel Bearings	7
25	BE1016X-01	5/8" Ceramic Wheel Bearing	3
26	WH3131X-05	24" Wheel Complete with 5/8" Bearings, Handrim & Hub Cap	5

Fig	Model	Description	Page
27	WH3121X-05	Axle and Nut for Rear Wheel	8
28	WH3122X-05	Handrim Screws & Spacers (set of 8)	7
29	CA3024X-01	4" Large Style Hub Cap	9
30	WH3124X-05	Black Nylon Handrim	7
35	AR5001X-05-CL	Desk Length Armrest, Color **	12
38	AR5002X-05	Skirtguard 15 1/4" x 5 1/2"	12
39A	AR5004L-05-CL	Flip Back Left Side Desk Length Arm, Complete with skirtguard, armrest & button lock, Color * **	11
39B	AR5004R-05-CL	Flip Back Right Side Desk Length Arm, Complete with skirtguard, armrest & button lock, Color * **	11
40	AR5010X-05	Flip Up Arm Hardware Only (nut, bolt, spacers)	12
41	CA8008X-01	Flip Up Arm Lock, Release Lever	12
42	AR5006L-05 AR5006R-05	Flip Up Arm Socket, Left * Flip Up Arm Socket, Right *	10
50	BR8008L-05 BR8008R-05	Brake Complete, Left * Brake Complete, Right *	8
51	BR8007X-01	Rubber Tip for Brake	9
60	FM1008X-05	Handle Grip	9
61	FM1009X-05	Kick Tube Cap	9
72	WH3125L-05 WH3125R-05	Rear Axle Bracket, Left * Rear Axle Bracket, Right *	8
80	FM1007L-05 FM1007R-05	Lower Main Frame, Left * Lower Main Frame, Right *	20
81	FM1016X-05	Back Cane	20
82	FM1004X-05	Frame Support Tube, 1 Each	21
83	FM1013X-05	Front Rigging Top Mount Bushing	15

* Determine Left or Right when sitting in the wheelchair
** Upholstery Colors: 22-Burgundy, 50-Green, 02-Dk. Blue